

Specimen Collection Guide


Women's Health




LIQUID-BASED CYTOLOGY & ANCILLARY TESTING (SWAB)

TESTS OFFERED	TAT	SurePath™	ThinPrep®	AveroSwab™	Test Codes
SurePath Pap Test with Imaging	48 – 72 hrs	◆			SUREPATH
Onclarity HPV with Extended Genotyping (HPV-16, 18, 31, 45, 33/58, 35/39/68, 51, 52, 56/59/66) <i>Genotyping included, no reflex needed</i>	24 – 48 hrs	◆		◆	MV00001
ThinPrep Pap Test with Imaging	2 – 5 days		◆		THINPREP
Onclarity HPV with Extended Genotyping (HPV-16, 18, 31, 45, 33/58, 35/39/68, 51, 52, 56/59/66) <i>Genotyping included, no reflex needed</i>	24 – 48 hrs*		◆	◆	MV00001
OUT-OF-THE-VIAL ADVANCED TEST OPTIONS					
Leukorrhea Panel (CT, NG, TV)	24 – 48 hrs†	◆	◆	◆	MQ0016-TF
<i>Chlamydia trachomatis</i>	24 – 48 hrs†	◆	◆	◆	MQ000004-TF
<i>Neisseria gonorrhoeae</i>	24 – 48 hrs†	◆	◆	◆	MQ000019-TF
<i>Trichomonas vaginalis</i>	24 – 48 hrs†	◆	◆	◆	MQ000032-TF
<i>Mycoplasma genitalium</i>	24 – 48 hrs†	◆	◆	◆	MQ000017-T
Herpes Simplex Virus 1 & 2	24 – 48 hrs	◆	◆	◆	MQ0007-TF
Herpes Simplex Virus 1	24 – 48 hrs	◆	◆	◆	MQ000033-TF
Herpes Simplex Virus 2	24 – 48 hrs	◆	◆	◆	MQ000034-TF
<i>Treponema pallidum</i> (primary Syphilis)	24 – 48 hrs	◆	◆	◆	MQ000023-TF
Aerobic Vaginitis Panel	48 – 72 hrs	◆	◆	◆	MQ0010-TF
Bacterial Vaginitis Panel	48 – 72 hrs	◆	◆	◆	MQ0001-TF
Candidiasis Panel	48 – 72 hrs	◆	◆	◆	MQ0017-TF
Cervicitis/Vaginitis Panel	48 – 72 hrs	◆	◆	◆	MQ0018-TF
Mycoplasma Panel	48 – 72 hrs	◆	◆	◆	MQ0012-TF
PID/Infertility Panel	48 – 72 hrs	◆	◆	◆	MQ0019-TF
Standard Panel	48 – 72 hrs	◆	◆	◆	MQ0013-TF
Ureaplasma Urealyticum	48 – 72 hrs	◆	◆	◆	MQ000024-TF
Vaginosis/Vaginitis Panel	48 – 72 hrs	◆	◆	◆	MQ0014-TF
Vaginosis/Vaginitis Extended Panel	48 – 72 hrs	◆	◆	◆	MQ0015-TF

◆ **SurePath™ Kit**
Stability: 4 weeks ambient

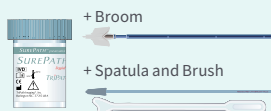

◆ **ThinPrep® Kit**
Stability: 4 weeks ambient

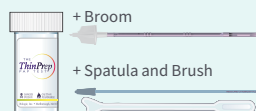

◆ **AveroSwab™**
Stability: 4 weeks ambient



Label each vial with two forms of patient ID. Follow kit instructions for collection of endocervical samples for Pap.

GYNECOLOGIC BIOPSY TESTING

TESTS OFFERED	TAT	Biopsy Kit	Test Codes
Cervical Biopsy	24 – 48 hrs	◆	8000
Endocervical Curettage	24 – 48 hrs	◆	8000
Vaginal Biopsy	24 – 48 hrs	◆	8000
Vulvar Biopsy	24 – 48 hrs	◆	8000
LEEP	24 – 48 hrs	◆	8000

◆ **10% Neutral Buffered Formalin Biopsy Kit**
Stability: Room temperature

Label with two forms of patient ID.

Obtain biopsy (ies) with care not to crush the specimen with forceps, hemostats, or other instruments. Cautery will cause heat artifact. Use a separate vial for each specimen.



OTHER MOLECULAR TESTS & CULTURES




TESTS OFFERED	Copan eSwab™	Test Codes
Routine Aerobic/Anaerobic Cultures	◆	M_AER_ANER
Group B Streptococcus (RT-PCR)	◆	STRPB
Yeast Culture	◆	M_YEAST
Throat Culture	◆	M_THROAT
Eye Culture	◆	M_EYE
Ear Culture	◆	M_EAR
Genital Culture	◆	M_AER_ANER (Genital)

◆ Copan eSwab™
Stability: 2 days ambient



Label each vial with two forms of patient ID. Follow kit instructions for collection of culture samples and include date and time of collection.


LIQUID-BASED CYTOLOGY & ANCILLARY TESTING (URINE)

TESTS OFFERED	TAT	Aptima® Urine Kit	BD Vacutainer®Kit	Urine Cytology Kit	Sterile Collection Cup	Test Codes
Leukorrhea Panel (CT, NG, TV)	24 – 48 hrs	◆				MQ0016-TF
<i>Chlamydia trachomatis</i> 	24 – 48 hrs	◆				MQ000004-TF
<i>Neisseria gonorrhoeae</i> 	24 – 48 hrs	◆				MQ000019-TF
<i>Trichomonas vaginalis</i> 	24 – 48 hrs	◆				MQ000032-TF
Urine Culture	2 – 5 days		◆			M_URINE
Urine Cytology	48 – 72 hrs			◆		6030
UTI Complete™ Panel	24 – 48 hrs				◆	MUP0001

◆ Aptima® Urine Kit
Stability: 1 week ambient



◆ BD Vacutainer® Urine Kit
Stability: 48 hours ambient

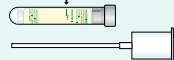

◆ Urine Cytology Kit
Sterile collection cup with
sample transported in
PreserveCyt, 2:1 ratio
Stability: 3 days ambient



◆ Sterile Collection Cup
Stability: 1 week ambient



Label each vial with two forms of patient ID. Follow kit instructions for collection of urine samples.

 Testing can be added up to 7 days from date of collection

 Testing can be added up to 30 days from date of collection

† Turnaround time increases to 48 – 72 hours if ordered with other panels

* Turnaround time will be increased if HPV is performed as a reflex test due to an abnormal Pap diagnosis

Turnaround Time is calculated based on the date sample is received in the Avero laboratory.

Reflex Tests: TAT will be extended on Pap tests if a reflex test is initiated based on the results of the ordered test.

Biopsies: TAT may be extended if abnormal results necessitate an external consult or additional stains.

**focused on
answers.**

6221 Riverside Drive, Suite 119
Irving, TX 75039
877.232.9924

3560 Meridian Street, Suite 101
Bellingham, WA 98225
360.527.4580

averodx.com

For asymptomatic patients, Medicare requires a letter of medical necessity. SurePath™ and Vacutainer® are registered trademarks of Becton, Dickinson and Company. Aptima® and ThinPrep® are registered trademarks of Hologic, Inc. UTM Universal Transport Medium® and eSwab® are registered trademarks of Copan Diagnostics, Inc.

Avero Diagnostics is a laboratory operations and pathology practice of Northwest Pathology, P.S. © 2023 Avero Diagnostics.
AV-16004-01-IRV REV 042023